**REF BFPF7/8-10**

## Prefilter

*For the pre-filtration of smoke plume generated during surgical procedures*

## INSTRUCTIONS

BFPF7/8 Prefilter can be used with any ViroSafe® Filter with a 7/8" (22 mm) port.

**PRECAUTIONS:**

1. Single use only. Remove and discard prefilter after each surgical procedure.
2. Before installing or removing any filter, disconnect system from electrical outlet.
3. Install a fluid collective device if excessive fluid is expected. Failure to install a fluid collection device could cause filter blockage and electrical damage.

**INSTRUCTIONS:**

1. For female ViroSafe® applications, Insert the BFPF7/8 Prefilter into the 7/8" (22 mm) port of the ViroSafe® Filter. The prefilter label should be facing toward user. Press in completely to assure complete sealing. For male port applications, use the provided 2" (5.08 cm) cuff to facilitate the prefilter insertion. Slide the cuff over the 7/8" (22 mm) male connection of the filter and then insert the prefilter into the cuff. Discard the cuff with the prefilter after single use.
2. Attach a 7/8" (22 mm) ID vacuum hose to the 7/8" (22 mm) male connector of the prefilter. Installation is complete.
3. At the end of the surgical procedure, disconnect smoke evacuator from electrical outlet. Carefully remove BFPF7/8 prefilter from the ViroSafe® Filter.
4. Dispose of BFPF7/8 prefilter in accordance with local codes. Used prefilters should be considered as potentially infectious waste.

European Commission requires that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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MDSS GmbH