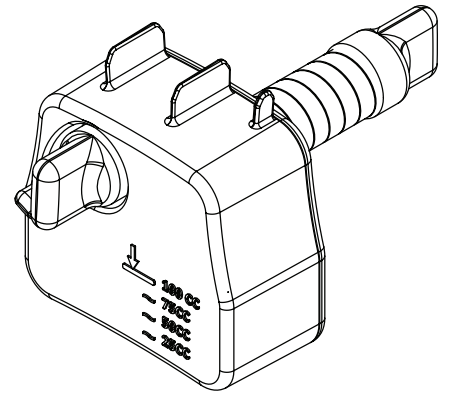
ENGLISH	1
FRANÇAIS	3
ITALIANO	5
DEUTSCH	7
ESPAÑOL	9
ΕΛΛΗΝΙΚΑ	11
PORTUGUÊS	13
PORTUGUÊS DO BRASIL	15
NEDERLANDS	17
DANSK	19
SVENSKA	21
SUOMI	23
NORSK	25
简体中文	27
日本語	29
POLSKI	31
TÜRKÇE	33
ROMÂNĂ	35
العربية	37
РУССКИЙ	39
HRVATSKI	41
ČEŠTINA	43
MAGYAR	45
LIETUVIŲ K.	47
SLOVENSKY	49
SLOVENŠČINA	51
한국어	53


# ViroSafe® Fluid Trap
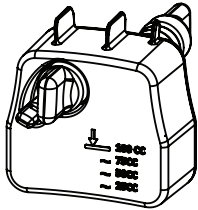
## Instructions For Use


Designed for use with Buffalo Filter ViroSafe® Filters except VS12D01 and VS12D02


en

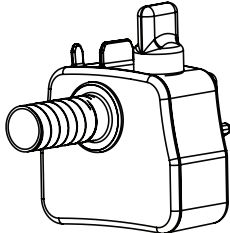1. Remove the fluid trap from its packaging.

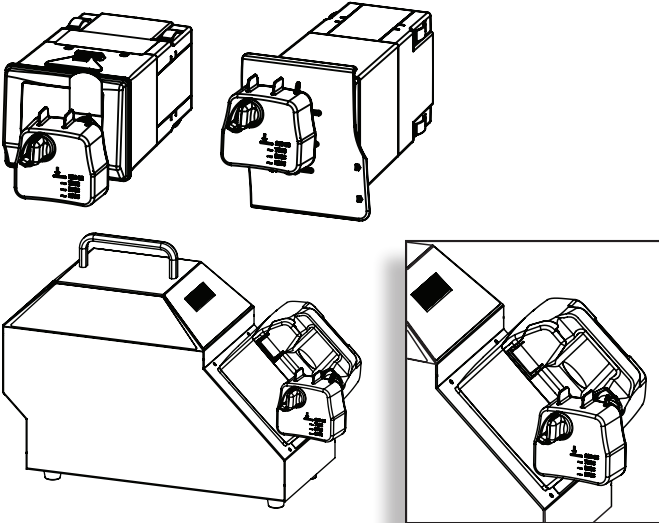
2. Remove rear cap (side without the fluid measurements) and place on appropriate cap holder on top of fluid trap.


3. Open the 22 mm (7/8 in) port on the ViroSafe® filter and insert the fluid trap, as shown, into the filter. Make certain the fluid trap is fully seated into the filter.

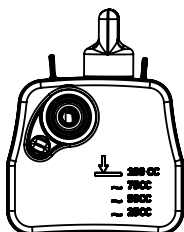
**CAUTION:** Ensure fluid trap is fully seated into the filter prior to capturing fluid.

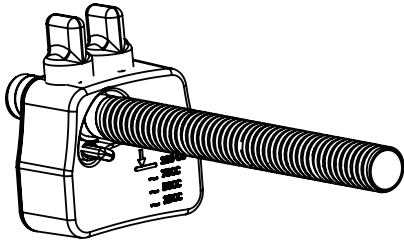4. Remove the appropriate cap on the front of the fluid trap based on the size of the tubing that will be used in the surgical application. Store the cap on the appropriate cap holder on top of fluid trap.

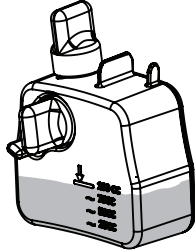
5. Insert the chosen tubing accessory into the appropriate port located on the fluid trap.


6. The fluid trap is equipped with measurement increments located on the front of the device to approximate the amount of fluid captured.

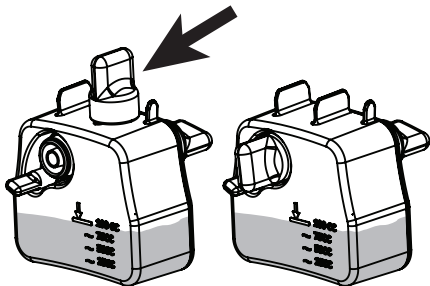
**WARNING:** Ensure that the fluid trap is not over-filled. Once the fluid level reaches 100cc, the fluid trap should be changed.

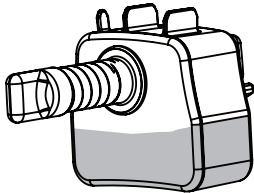7. Once filled, or at the end of each procedure, remove the tubing accessory and place the appropriate cap stored on the top of the fluid trap onto the open port.

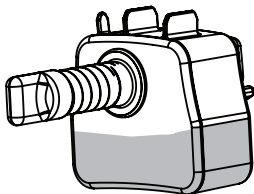8. Carefully remove the fluid trap from the filter and replace the cap on the back of the fluid trap.

9. Discard the used fluid trap per facility protocol.

**WARNING:** The fluid trap is completely disposable and should be considered biohazardous after use. Please dispose of according to your local codes or regulations and hospital policy. This fluid trap may be disposed of or incinerated, whichever is appropriate for your institution.

**WARNING:** The fluid trap should only be used by a trained individual.

For single use only. Reuse, reprocessing, or re-sterilization may result in device failure or cross contamination. European Commission requires that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**CONMED**  
**BUFFALO FILTER**

CONMED Corporation

525 French Road

Utica, NY 13502-5994 USA

Customer Service: 1-866-426-6633

USA FAX: (727) 399-5256

International FAX: +1 (727) 397-4540

email: [customerexperience@conmed.com](mailto:customerexperience@conmed.com)

[www.conmed.com](http://www.conmed.com)

©2021 CONMED Corporation. All Rights Reserved.

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