

ClearViewTM

Anti-Fog Scope Warmer



Pre-heats scope to body temperature

No electricity or hot water

One click to activate



supporting surgical outcomes


ClearView™

Anti-fog scopewarmer




Instructions for use

1. LAY SCOPES IN WARMER

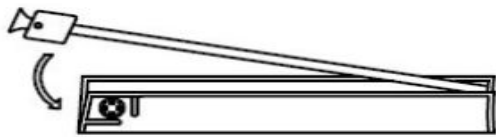

2. FLEX DISK TO START WARMING

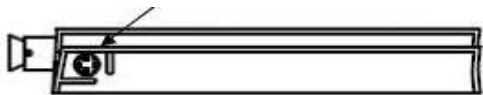

3. WARM FOR USE


Note: best to use scopewarmer 15 minutes before a scope is needed for deep warming.

Clearview™ stays warm up to 2 hours.

Cover with sterile towel to preserve warmth.

References

Clearview scopewarmer generates a significant higher difference during the complete process of pre-warming the endoscope. Clearview exchanges 75% more energy at the start compared to a warm water bath. Over time the diffusion capacity of a warm bath reduces to only 12% of the heat flux from the Clearview scopewarmer*. This significant difference ensures that enough energy is diffused during the process to ensure the whole endoscope, including the optic materials inside, are pre-warmed to body temperature.

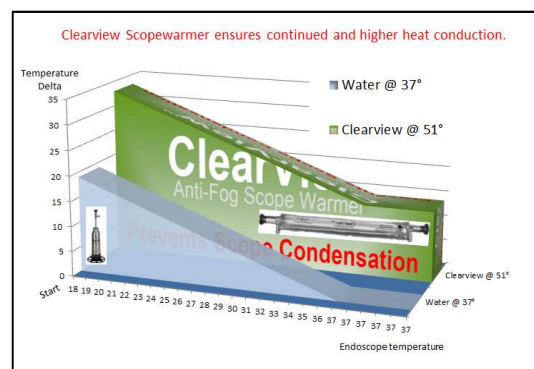

* Data on file

supporting surgical outcomes

2MEDICAL-europe BV The Netherlands REF: 31SW01